

# DiaClean™ Diamond Knife and Delicate Instrument Cleaning System

## Directions for Use

### Tray Set-Up Instructions

At the beginning of each surgical day, set up a new disposable cleaning tray as follows:

1. Remove a new tray from the dispenser box.
2. Peel back and dispose of the foil covering, exposing the cleaning compartments.



### Step 1

Fully expose the blade and gently insert it directly into the blue colored wet cleaning pad located in the larger of the three compartments. Gently pass it through the cleaning pad in a cutting fashion, creating about a 1/2 inch incision in the pad. Several gentle passes without removing the blade from the pad will remove tissue and other debris from the blade surface. Remove and examine the blade. Repeat step until blade is visibly clean.




### Step 2

To rinse the DiaClean™ solution from the blade and to remove residual debris, repeat Step 1, creating a 1/4 inch incision in both of the two smaller compartments located just above the larger cleaning compartment. Always start with the rinse compartment located to your left side and perform a final rinse in the compartment located to the right.



### Step 3

Retract the blade and continue with your knife manufacturer's care, cleaning and sterilization procedure. In the case of Diamatrix products, Step 4.3 of DTF 014 – Diamatrix Diamond Knives Directions for Use.

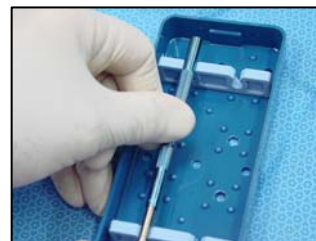

### Step 4

At the end of the surgical day, or when the cleaning or rinse solutions and pads become visibly soiled, the used tray along with its contents should be disposed of following local regulations.



Scan the Code to view an instructional video for the DiaClean™ Delicate Instrument Cleaning System.

If you do not have a code reader on your mobile device, search "ScanLife" from your app store.

For technical assistance or questions regarding care, cleaning and sterilization of our products, please contact us at 800-867-8081 (US Toll Free) or 281-367-8081.  
www.diamatrix.com

DFT 0015 Rev A