1.0 Intended Use:

The Xpand^{NT} Iris Speculum is a device for the mechanical dilation of the pupil during cataract surgery. It is generally used when the pupil will not dilate via normal means due to chronic parasympathomimetic drug use, scarring or trauma, or if the patient has an unusually small pupil.

2.0 Precautions:

- Use of the device is restricted to qualified physicians and should only be used by those physicians familiar with techniques employed during ophthalmic surgical procedures. As with all medical devices, sound professional judgment must always be used in determining if a device is fit for surgical use and the physician has the sole responsibility in making that determination.
- All persons using this device should be knowledgeable in the use and handling of surgical instruments, accessories and related equipment.
- Do not use if damaged (broken, misshaped, etc) and do not attempt to repair any damage yourself.

3.0 Device Description:

The Xpand^{NT} Iris Speculum is made from a medical grade titanium alloy and the Injector is composed of medical grade Polyoxyethylene copolymer (body) and stainless steel (cannulated lumen). The Xpand^{NT} Iris Speculum is pre-loaded inside the Injector. The device is packaged sterile, inside a Tyvek[®] sealed tray.

4.0 Instructions for Use:

Once the device has been removed from the tray:

4.1 Inserting the Xpand^{NT} Iris Speculum:

- 4.1.1 Introduce Ophthalmic Viscosurgical Device (OVD) into the anterior chamber (AC) and under the iris rim.
- 4.1.2 Slide the cannulated lumen of the Injector into the AC via the primary incision and position the cannulated lumen to approximately mid-pupil.
- 4.1.3 Advance the injector plunger so the distal foot of the Xpand^{NT} Iris Speculum emerges from the cannulated lumen and engages the distal iris rim.
- 4.1.4 While keeping the distal iris rim engaged, continue to advance the plunger, allowing the two lateral feet to emerge from the cannulated lumen and engage the iris rim while the proximal foot rests on the anterior iris.
- 4.1.5 Using the Williamson Xpand Manipulator (item no.24277) or similar device, engage the proximal foot and place it on the proximal iris rim.

Note: If the proximal and distal feet are aligned with the primary incision the Xpand^{NT} Iris Speculum is in the proper position.

4.2 Removing the Xpand^{NT} Iris Speculum from the AC:

- 4.2.1 Following lens implantation and prior to removal of the OVD, using a push/pull type manipulator, such as the Williamson Xpand Manipulator, disengage all four feet.
- 4.2.2 Using the same or similar push/pull device, engage the proximal foot and remove the device from the AC via the primary incision.
- 4.2.3 Alternatively, after step 4.2.1, the injector can be re-introduced into the AC through the primary incision and the device can be re-engaged and drawn back into the injector, which is then removed from the AC via the primary incision.
- 4.2.4 Dispose of the Xpand^{NT} Iris Speculum, including the Injector.

5.0 Contents:

- Four (4) Xpand^{NT} Iris Speculum Devices
- One (1) Instructions for Use

	Diamatrix Ltd, 210 Nursery Road, The Woodlands, Texas 77380 USA www.diamatrix.com
	Advana Limited, Tower Business Centre, 2nd Flr., Tower Street, Swatar, BKR 4013 Malta
	Do Not Re-Sterilize 
	Available By Prescription Only
	Consult These Directions for Use Prior to Using This Device
	Single Use Only
	Do Not Use if Packaging is Damaged
	Sterilized via Irradiation

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Advena Limited, Tower Business Centre, 2 nd Flr., Tower Street, Swatar, BKR 4013 (Malta)